


Shimadzu Mobile DaRt Evolution

Replacement Tube Installation and Adjustment

Step #1 Opening to normal operation mode

Remove existing X-Ray tube and Install New Replacement

1. Turn on the control panel key switch (top right). It can take a few seconds for the computer to open. The KV and mAs display **004** **132** windows will begin flashing software number and letter codes and the Registration Key  button will also be flashing.

NOTE: A 4 digit password may be required to **P** **_ _ _** enter the system.

2. Press the Registration Key  button.

The letter [P] will be displayed in the KV window.

3. Using the numeric keypad enter the 4 digit password, (left side of the control panel).

4. Press the registration key button  again to enter normal operation mode.

